



# IV Clear™

## Antimicrobial Clear Silicone Adhesive Securement Dressing with Chlorhexidine and Silver

**APPLICATION INSTRUCTIONS:** As with any medical device, always refer to the full instructions for use prior to application.

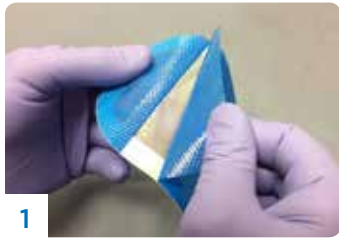

1 Hold the flap of the top blue liner and peel off.

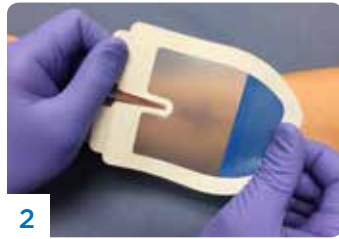

2 Place the exposed adhesive on the skin and smooth the dressing over the skin to ensure good adhesion.



3 Hold the flap of the remaining blue liner and peel off. Smooth the dressing over the skin towards the edges to ensure good adhesion.



4 Holding on to the tab of the white paper backing, carefully peel off the backing.



5 Smooth the dressing towards the edges to ensure good adhesion.



6 After dressing has been applied, apply the tape strip(s) to further secure dressing, IV tubing, or to stabilize catheter as good nursing practices dictate.



7 The optional tape strips provided should not be used on patients with known hypersensitivity or allergy to acrylic adhesives.



8 Record the dressing change on the documentation strip in accordance with your facility's protocol.

**APPLICATION VIDEO:** [https://youtu.be/5n\\_DphZxWCA](https://youtu.be/5n_DphZxWCA)



### IV Clear™

Antimicrobial Clear Silicone Adhesive Securement Dressing

1-877-711-6055 | [ivclear@covalon.com](mailto:ivclear@covalon.com)

REF TWBD1014

#### SITE PREPARATION

- Read precautions and warnings in package insert prior to use
- Prepare the site according to your facility's protocol.
- Clipping hair at the site may improve dressing adhesion.
- Allow all aseptic preparations to dry completely before applying IV Clear™ to ensure good adhesion.

#### FREQUENCY OF CHANGE

IV Clear™ provides antimicrobial dressing protection for up to seven days, and will remain in place depending on the condition of the site.

#### CARE AND REMOVAL

- Inspect the dressing daily and change as per facility protocol
- The dressing should be changed as necessary:
  - If the dressing becomes loose, soiled or compromised in any way.
  - If the site is obscured or no longer visible.
  - If there is visible drainage.
- Remove documentation label and securement tape strip(s) from dressing. To remove the dressing, gently lift one edge of the dressing and slowly peel away.

**covalon**  
TECHNOLOGIES LTD.