


IV Clear™

Antimicrobial Clear Silicone Adhesive Securement Dressing with Chlorhexidine and Silver

APPLICATION INSTRUCTIONS: As with any medical device, always refer to the full instructions for use prior to application.

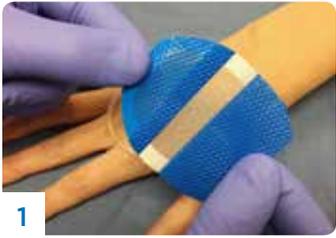

1 Hold the flap of the top blue liner and peel off.



2 Place the exposed adhesive on the skin and smooth the dressing over the skin to ensure good adhesion.




3 Hold the flap of the remaining blue liner and peel off. Smooth the dressing over the skin towards the edges to ensure good adhesion.



4 Holding on to the tab of the white paper backing, carefully peel off the backing.



5 Smooth the dressing towards the edges to ensure good adhesion.

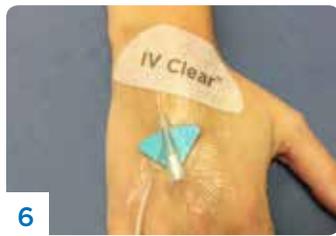

6 After dressing has been applied, apply the tape strip(s) to further secure dressing, IV tubing, or to stabilize catheter as good nursing practices dictate.



7 The optional tape strips provided should not be used on patients with known hypersensitivity or allergy to acrylic adhesives. Record the dressing change on the documentation strip in accordance with your facility's protocol.

APPLICATION VIDEO: <https://youtu.be/tKPkElglIAU>



IV Clear™

Antimicrobial Clear Silicone Adhesive Securement Dressing

1-877-711-6055 | ivclear@covalon.com

REF TWBD1020

SITE PREPARATION

- Read precautions and warnings in package insert prior to use
- Prepare the site according to your facility's protocol.
- Clipping hair at the site may improve dressing adhesion.
- Allow all aseptic preparations to dry completely before applying IV Clear™ to ensure good adhesion.

FREQUENCY OF CHANGE

IV Clear™ provides antimicrobial dressing protection for up to seven days, and will remain in place depending on the condition of the site.

CARE AND REMOVAL

- Inspect the dressing daily and change as per facility protocol
- The dressing should be changed as necessary:
 - If the dressing becomes loose, soiled or compromised in any way.
 - If the site is obscured or no longer visible.
 - If there is visible drainage.
- Remove documentation label and securement tape strip(s) from dressing. To remove the dressing, gently lift one edge of the dressing and slowly peel away.

covalon
TECHNOLOGIES LTD.